

Eyewash in case of foreign objects

This eyewash is a sterile 0.9 % sodium chloride solution that eases discomfort and prevents following complications caused by foreign bodies in the eyes

PRODUCT FACTS & BENEFITS

- Effectively removes dust, dirt, and contamination in the eyes
- Easy to handle easily applied by the injured person or with assistance
- Flexible solutions bottles for mobile use and stations for wallmounting
- Ergonomic eyecups providing a soft and pleasant flow
- Wide application from industrial workplaces to laboratories and first aid kits
- High functional reliability bottles comply with to EN 15154-4
- Shelf life of 3.5 years for sealed bottles



The product is
CE marked according
to the Guidelines for
Classification of Medical
Device Directive (MDD
93/42/EEC).

info@plum.eu +45 6916 9600



Combine Plum Eyewash 0.9% Sodium Chloride with Plum Eyewash pH Neutral 4.9% Phosphate for workplaces with a risk of chemical accidents

Know your chemicals and ensure the best suited first aid

The cause of an accident determines which eyewash is required. When an accident occurs, it is essential to determine whether it is caused by either a foreign object or by acids or alkali - and if possible, which pH value the substance in question holds.

When pH is either strongly acidic or alkalic, start rinsing with Plum Eyewash pH Neutral 4.9% Phosphate until the bottle is empty. Discard the bottle and rinse with Plum Eyewash 0.9% Sodium Chloride until you reach a doctor or the emergency room.

When pH is neutral, rinse with Plum Eyewash 0.9% Sodium Chloride until the bottle is empty and discard the bottle after use.



ERGONOMIC EYECUPS

The bottles are equipped with a selection of ergonomic eyecups to ensure that the liquid can be applied correctly and in an even flow. Squeeze the bottle gently to generate a soft and pleasant flow.

EYECUP



DUO EYECUP

Flexible, ergonomic

eyecup that keeps

the eye lid open

during use



Flexible, ergonomic eyecup that allows rinsing of both eyes at the same time

AVAILABLE SIZES

Art. no.	YECUP	4691	4604	4707	DUO	4861	4800
Size		200 ml	500 ml	1000 ml		500 ml	1000 ml
Packaging	Ш	10 x 200 ml	12 x 500 ml	6 x 1000 ml	Ш	6 x 500 ml	6 x 1000 ml

info@plum.eu +45 6916 9600

