


Thermo Scientific™ *Brilliance*™ VRE

User Guide

presumptive identification of Vancomycin-resistant Enterococci (VRE)

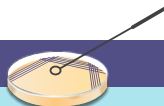

Inoculate plate. Incubate at 35°C +/-2°C for 24 hrs.

Read results.



Positive



Indigo to purple
E. faecium



Light blue
E. faecalis

Negative



Negative plates should
be re-incubated for an
additional 24 hrs.

Identifications are presumptive and should be confirmed. Please note, organisms with an atypical enzyme pattern may give anomalous reactions on Thermo Scientific™ *Brilliance*™ VRE Agar.

thermoscientific.com/preparedmedia

© 2014 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific and its subsidiaries.

Thermo
SCIENTIFIC

A Thermo Fisher Scientific Brand