www.thermofisher.com

C-F-C Selective Supplement

REF SR0103E

EN

* This instructions for use (IFU) document is intended to be read in conjunction with the IFU for Pseudomonas Agar Base (Dehydrated). Product codes CM0559B/R/T

Intended Use **IVD**

C-F-C Supplement (SR0103E) is a selective supplement that is intended to be used in the preparation of Pseudomonas Agar Base (Dehydrated) (CM0559B, CM0559R and CM0559T) for the isolation of *Pseudomonas* spp. from a wide range of clinical samples including urine, wound swabs, and sputum. The devices can also be used for testing food, water, and environmental samples. Pseudomonas C-F-C Supplement (SR0103E) is recommended for the selective isolation of *Pseudomonas* spp. generally.

C-F-C Supplement (SR0103E) is used in a diagnostic workflow to aid clinicians in determining potential treatment options for patients suspected of having Pseudomonas infections. The device is for professional use only, is not automated, nor is it a companion diagnostic.

Summary and Explanation

Pseudomonas species are motile, gram negative bacteria that are straight or curved in appearance¹. *Pseudomonas* species can cause a variety of infections in humans, ranging from lungs (pneumonia), wound infections, septicemia, urinary tract and gastrointestinal infections².

Pseudomonas aeruginosa is a non-fermentive, strictly aerobic, catalase and oxidase positive gram-negative rod. It is widely distributed in soil and water and may cause disease in animals, plants and insects. *P. aeruginosa* is known as the key respiratory pathogen in patients with cystic fibrosis³. In addition to it, people with, chronic obstructive pulmonary disease (COPD), pregnant ladies and people with a weakened immune system are most affected by the *Pseudomonas* infection². *P. aeruginosa* are known to cause a condition known as pyelonephritis particularly in patients with catheters⁴. In general infections caused by *Pseudomonas aeruginosa* are treated with antibiotic. However, with an increase in the number multi drug resistant strains of *Pseudomonas aeruginosa* presumptive identification of the strains are highly important. For the identification of best antibiotics to treat the infection clinical specimen (culture) to a laboratory which are grown against a set of antibiotics like fluoroquinolones².

Principle of Method

C-F-C Supplement (SR0103E) when used with the Pseudomonas Agar Base (Dehydrated) (CM0559B, CM0559R and CM0559T), is used for the isolation of *Pseudomonas* spp., which is achieved through the inclusion of cetrime, fucidin and cephalosporin to inhibit the growth of competing flora and improve selectivity for the identification of *Pseudomonas* spp.

Typical Formula

	Milligrams per 500ml
Cetrime	5.0mg
Fucidin	5.0mg
Cephalosporin	25.0mg

Materials Provided

SR0103E: 10x freeze dried vials each for 500ml of medium

Materials Required but Not Supplied

1. Pseudomonas Agar Base (dehydrated) product code CM0559B/R/T
2. Inoculating loops
3. Swabs
4. Collection containers
5. Incubators
6. Quality control organisms

Storage

- Store product in its original packaging at 2–8°C until used.
- The product may be used until the expiry date stated on the label.
- Store away from light.
- Allow product to equilibrate to room temperature before use.
- Do not incubate prior to use.

Warnings and Precautions

Danger

- May cause allergy or asthma symptoms or breathing difficulties if inhaled
- May cause an allergic skin reaction
- Causes serious eye irritation
- May cause respiratory irritation
- Causes skin irritation
- Avoid breathing dust/fume/gas/mist/vapours/spray
- IF INHALED: If breathing is difficult, remove person to fresh air and keep at rest in a position comfortable for breathing
- IF ON SKIN: Wash with plenty of soap and water
- If Skin irritation or rash occurs: Get medical advice/attention
- Call a POISON CENTER or doctor/physician if you feel unwell
- Avoid release to the environment
- DO NOT USE SUPPLEMENTS IF YOU ARE PREGNANT OR CONTEMPLATING PREGNANCY.
- Rehydrate all supplements in Safety Cabinet, wear disposable gloves.
- Supplements may be dispensed on the bench after rehydration, wear disposable gloves and safety glasses.
- Disposable gloves must be worn for rehydrating, dispensing and disposing of ALL Supplements (i.e. wear disposable gloves whilst handling supplements).
- All waste (i.e. rehydrated supplement) must be emptied into the HAZARDOUS WASTE container situated in the Microbiology laboratory.
- The vial and rubber bung must be rinsed into the HAZARDOUS WASTE container.
- The metal lid and rinsed rubber bung should be placed in the waste bin.
- The rinsed glass vial should then be placed in the glass bin.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

Specimen should be collected and handled following local the recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI) ID 17, B 11, B 41 and B 57.

Procedure


1. Allow supplement to equilibrate to room temperature before use.
2. Aseptically add 2ml of 1:1 ethanol:sterile distilled water to 1 vial and mix gently to dissolve.
3. Aseptically add the vial contents to 500ml of sterile *Pseudomonas* Basal Medium, Prepared from *Pseudomonas* Agar Base (CM0559) plus 5ml glycerol and cooled to 50°C.
4. Mix well and aseptically dispense into sterile containers.

Bibliography


- 1) Public Health England 'Identification of *Pseudomonas* species and other NonGlucose Fermenters.' ID1713 24.04.2015.
<https://www.gov.uk/government/publications/smi-id-17-identification-of-glucose-non-fermenting-gram-negative-rods>
- 2) Centers for Disease control and Prevention (CDC). 2019. '*Pseudomonas aeruginosa* in Healthcare Settings
<https://www.cdc.gov/hai/organisms/pseudomonas.html>
- 3) Langan, K. M., Kotsimbos, T., & Peleg, A. Y. (2015). Managing *Pseudomonas aeruginosa* respiratory infections in cystic fibrosis. *Current Opinion in Infectious Diseases*, 28(6), 547–556.
- 4) Newman, J. W., Floyd, R. v, & Fothergill, J. L. (2017). The contribution of *Pseudomonas aeruginosa* virulence factors and host factors in the establishment of urinary tract infections. *FEMS Microbiology Letters*, 364(15).

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date

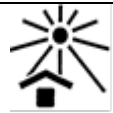	Keep away from sunlight
	Do not re-use
	Consult instructions for use or consult electronic instructions for use
	Contains sufficient for <n> tests
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Importer - To indicate the entity importing the medical device into the locale. Applicable to the European Union
	Made in the United Kingdom

©2023 Thermo Fisher Scientific Inc. All rights reserved.
All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

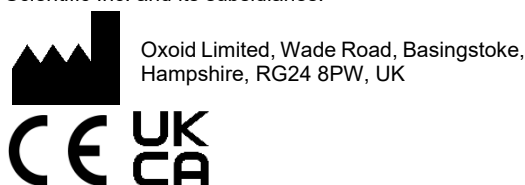For technical assistance please contact your local distributor.

Revision information

Version	Date of issue and modifications introduced
2.0	2023-11-09.

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Typisk formel

	Milligram pr. 500ml
Cetrimid	5,0 mg
Fucidin	5,0 mg
Cephalosporin	25,0 mg

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©2022 Thermo Fisher Scientific Inc. Med ensamrätt.
Alla övriga varumärken tillhör Thermo Fisher Scientific Inc. och dess dotterbolag.
Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, UK