www.thermofisher.com

Modified Preston Campylobacter Supplement

EN

REF SR0204E

*This instructions for use (IFU) document is intended to be read in conjunction with the IFU for Campylobacter Agar Base (Dehydrated) (product code: CM0689B)

Intended Use

Modified Preston Campylobacter Selective Supplement (SR0204E) is a selective supplement intended to be used in the preparation of Preston Campylobacter selective medium for the isolation of *Campylobacter* species from faecal samples. When Campylobacter Agar Base (CM0689B) is supplemented with Modified Preston Campylobacter Selective Supplement (SR0204E) it is intended to be used in a diagnostic workflow to aid clinicians in determining potential treatment options for patients suspected of having bacterial gastroenteritis.

The device is for professional use only, is not automated and nor is it a companion diagnostic.

Summary and Explanation

Campylobacter species are mobile, Gram-negative rods that are 0.5 to 8µm in length with a curved, spiral or S shape¹. *Campylobacter* spp. are considered nutritionally fastidious and grow under anaerobic or microaerobic conditions (5-10% of O₂ and CO₂), with some strains requiring a hydrogen-enriched atmosphere¹. The optimum growth temperature for these thermophilic organisms is between 37°C and 42°C¹. *Campylobacter jejuni* and *Campylobacter coli* are two well-known clinically relevant strains of *Campylobacter* species that are known to cause illness in humans¹. *C. coli* is a spiral shaped and *C. jejuni* is spiral shaped¹. *Campylobacter* spp. can be isolated from a variety of clinical sample types, including; blood, faeces, cerebrospinal fluid, the intestinal tract, gall bladder, brain abscess, urine, oral cavities and wounds¹.

Principle of Method

Modified Preston Campylobacter Selective Supplement (SR0204E) is a selective supplement intended to be used in the preparation of Preston Campylobacter selective medium for the isolation of *Campylobacter* species. When Campylobacter Agar Base (CM0689B) is supplemented with Modified Preston Campylobacter Selective Supplement (SR0204E) nutrients present in Campylobacter Agar Base (CM0689B) support the growth of *Campylobacter* species. The isolation of *Campylobacter* species is achieved through the critical components found in Modified Preston Campylobacter Selective Supplement (SR0204E). Polymyxin B and Rifampicin are broad spectrum antibiotics which inhibit the growth of Gram- negative bacteria. Trimethoprim disrupts the synthesis of nucleic acids and proteins; and displays bacteriostatic action on most Gram-positive and Gram-negative organisms. Amphotericin B is also present which acts as an anti-fungal agent. This antibiotic/antifungal mix inhibits the growth of non-target Gram-negative bacteria, allowing the target organism *Campylobacter* to grow.

Typical Formula

	Milligrams per 500ml
Polymixin B	2500IU
Rifampicin	5.0mg
Trimethoprim	5.0mg
Amphotericin B	5.0mg

Materials Provided

SR0204E: 10x freeze dried vials each for 500ml of medium

Materials Required but Not Supplied

1. CM0689B: 500g of dehydrated Campylobacter Agar Base
2. AnaeroJar 2.5L (AG0023A) AnaeroJar 3.5L (AG0035A)
3. CampyGen 2.5L Sachet (CN0025A) CampyGen 3.5L Sachet (CN0035A)
4. Inoculating loops, swabs, collection containers
5. Incubators
6. Quality control organisms

Storage

- Store product in its original packaging between 2°C and 8°C.
- Keep container tightly closed.
- The product may be used until the expiry date stated on the label.
- Protect from moisture.
- Store away from light.
- Allow reconstituted product to equilibrate to room temperature before use.

Warnings and Precautions

Signal Word: Warning

Hazard Statements
H315 - Causes skin irritation

H319 - Causes serious eye irritation

Precautionary Statements

P302 + P352 - IF ON SKIN: Wash with plenty of soap and water

P280 - Wear face protection

P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing

Each vial is for single use. Do not re-use.

For in vitro diagnostic use only.

For professional use only.

Inspect the product packaging before first use.

Do not use the product if there is any visible damage to the packaging or vials.

Do not use the product beyond the stated expiry date.

Do not use the device if signs of contamination are present.

Do not use the device if the colour has changed or there are other signs of deterioration.

It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

There are no specimen collection and/or transport materials provided with this device. Specimens should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI) ID 23, S 7 and Q5.

Procedure

1. Allow supplement to equilibrate to room temperature before use.
2. Aseptically add the vial contents to 500ml of sterile Campylobacter Agar Base (CM0689B) prepared as directed and cooled to 50°C.
3. Mix well and aseptically dispense into sterile containers.

Bibliography

1. Public Health England. 2018. "Identification of Campylobacter Species." UK Standards for Microbiology Investigations. <https://www.gov.uk/government/publications/smi-id-23-identification-of-campylobacter-species>.

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date
	Keep away from sunlight
	Do not re-use
	Consult instructions for use or consult electronic instructions for use

	Contains sufficient for <n> tests
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Importer - To indicate the entity importing the medical device into the locale. Applicable to the European Union
Made in the United Kingdom	Made in the United Kingdom

©2022 Thermo Fisher Scientific Inc. All rights reserved. All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, UK

For technical assistance please contact your local distributor.

Revision information

Version	Modifications introduced
3.0	2024-07-16

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Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, Storbritannien