www.thermofisher.com

Chloramphenicol Selective Supplements

[REF] SR0078E, SR0078H*

*These Instructions for Use (IFU) are intended to be read in conjunction with the IFU for Sabouraud Dextrose Agar (Dehydrated) (CM0041B, CM0041K, CM0041R, CM0041T).

Intended Use

Chloramphenicol Selective Supplements (SR0078E, SR0078H) are intended to be used in the preparation of Sabouraud Dextrose Agar (Dehydrated) (CM0041B, CM0041K, CM0041R, CM0041T) for the isolation and maintenance of dermatophytes, other fungi, and yeasts from skin, hair, nails, genital, respiratory, and urine samples.

Chloramphenicol Selective Supplements, when used with Sabouraud Dextrose Agar, are used in a diagnostic workflow to aid clinicians in determining treatment options for patients suspected of having fungal infections.

The devices are for professional use only, are not automated nor are companion diagnostics.

Summary and Explanation

Sabouraud Dextrose Agar is suitable for cultivation and differentiation of fungi and may be used in the place of the American medium Hodges.¹ The fungi maintain their typical cultural appearance and thus may be readily identified according to the standard macroscopic characters described by Sabouraud.² The medium is often used with antibiotics for the isolation of pathogenic fungi and *Candida albicans* from material containing large numbers of other fungi and bacteria.³⁻⁸

Principle of Method

Sabouraud Glucose Agar contains mycological peptone and glucose to support growth of fungi. It is slightly acidic which helps to inhibit bacteria whilst being favourable to fungi. The addition of antimicrobial agents can increase the selectivity of the medium. Chloramphenicol is a bacteriostatic antimicrobial with a broad spectrum of activity against both Gram-positive and Gram-negative bacteria.

Typical Formula

SR0078E: 50 mg Chloramphenicol
SR0078H: 200 mg Chloramphenicol

Materials Provided

SR0078E: 10 freeze dried vials each for supplementing 500 ml of medium
SR0078H: 10 freeze dried vials each for supplementing 2L of medium

Materials Required but Not Supplied

- Sabouraud Dextrose Agar (CM0041B, CM0041K, CM0041R, CM0041T, BO0408T, BO0408M, BO0408V, or BO0342E.)
- Inoculating loops, swabs, collection containers
- Incubators
- Quality control organisms

Storage

- Store product in its original packaging between 10°C and 30°C.
- Keep container tightly closed.
- The product may be used until the expiry date stated on the label.
- Store away from light.
- Allow product to equilibrate to room temperature before use.

Warnings and Precautions

Danger

- May cause cancer
- Wear protective gloves/ protective clothing
- IF ON SKIN: wash with plenty of soap and water
- If experiencing respiratory symptoms: Call a POISON CENTRE or doctor/ physician
- IF IN EYES: Rinse immediately with plenty of water, also under the eyelids, for a least 15 minutes. Get medical attention if irritation persists
- IF INGESTED: Clean mouth with water and drink afterwards plenty of water. Get medical attention
- IF INHALED: Remove person to fresh air and keep comfortable for breathing
- Each vial is for single use. Do not re-use.
- For in vitro diagnostic use only.
- For professional use only.
- Inspect the product packaging before first use.
- Do not use the product if there is any visible damage to the packaging or vials.
- Do not use the product beyond the stated expiry date.
- Do not use the device if signs of contamination are present.
- Do not use the device if the colour has changed or there are other signs of deterioration.
- It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products
- DO NOT USE SUPPLEMENTS IF YOU ARE PREGNANT OR CONTEMPLATING PREGNANCY.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

Specimen should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI) B 29 (Public Health England, 2020).

Procedure

SR0078E – supplement for 500 ml of medium

- Aseptically add 3 ml of ethanol to one vial of SR0078E.
- Mix gently to dissolve.
- Add vial contents to 500 ml of agar base prepared as directed.*
- Cool to 50°C.
- Mix well and aseptically dispense into sterile Petri dishes.

*Supplement should be aseptically added post-sterilization.

SR0078H – supplement for 2 litres of medium

- Aseptically add 12 ml of ethanol to one vial of SR0078H.
- Mix gently to dissolve.
- Add vial contents to 2 litres of agar base prepared as directed.*
- Cool to 50°C.
- Mix well and aseptically dispense into sterile Petri dishes.

*Supplement should be aseptically added post-sterilization.

Bibliography

1. Hodges R. S. (1928) *Arch. Derm. Syph., New York*, 18. 852.
2. Sabouraud R/ (1910) '*Les Teignes*', Masson, Paris.
3. Georg Lucille K., Ajello L. and Papageorge Calomira (1954) *J. Lab. Clin. Med.* 44. 422-428.
4. Ajello Libero (1957) *J. Chron. Dis.* 5. 545-551.
5. Williams Smith H. and Jones J.E.T. (1963) *J. Path. Bact.* 86. 387-412.
6. Hantschke D. (1968) *Mykosen.* 11. 113-115.
7. Dolan C.T. (1971) *Appl. Microbiol.* 21. 195-197.
8. Pagnano J., Levin J.G. and Trejo W. (1957-58) *Antibiotics Annual* 1957-58, 137-143.

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limitation (storage temp.)
	Use by (expiration date) YYYY-MM
	Keep away from sunlight
	Do not re-use
	Consult Instructions for Use
	Contains sufficient for <n> tests

	Do not use if packaging damaged
	Manufacturer
	Authorized representative in the European Community
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
Made in the United Kingdom	Made in the United Kingdom

©2022 Thermo Fisher Scientific Inc. All rights reserved. All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, UK

For technical assistance please contact your local distributor.

Revision information

Version	Date of issue and modifications introduced
2.0	2022-12-02

www.thermofisher.com

Chloramphenicol Selective Supplements

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©2022 Thermo Fisher Scientific Inc. Med ensamrätt.
Alla övriga varumärken tillhör Thermo Fisher Scientific Inc. och dess dotterbolag.