MacConkey Agar No.3

EN

REF CM0115B, CM0115K, CM0115M, CM0115Q, CM0115R, CM0115T, CM0115V & CM0115W

Intended Use

The MacConkey Agar No. 3 (CM0115B/K/M/Q/R/T/V/W) devices are selective media for the differentiation between lactose and non-lactose fermenters with the inhibition of Gram-positive cocci from clinical samples (e.g. faecal, blood, bile, pus, skin, mouth, etc.). Can also be used for testing food and water samples.

The MacConkey Agar No. 3 (CM0115B/K/M/Q/R/T/V/W) devices are intended to be used in a diagnostic workflow to aid clinicians in determining potential treatment options for patients suspected of having bacterial infections.

The devices are for professional use only, are not automated, and nor are they companion diagnostics.

Summary and Explanation

Microorganisms exist in a wide range of ecological niches, including environmental bacteria, commensals, and human pathogens. Prominent Gram-negative human pathogens include *Escherichia coli*, *Salmonella* species, *Shigella* species and *Proteus mirabilis*. Prominent Gram-positive human pathogens include *Staphylococcus aureus*, and *Enterococcus faecalis*. These organisms are associated with a range of different infections, with some causing high morbidity and mortality in immunocompromised individuals.

Principle of Method

Due to the inclusion of a specially prepared fraction of bile salts in addition to crystal violet, the MacConkey Agar No.

3 is a highly selective medium which gives improved differentiation between coliforms and lactose and non-lactose fermenters while Gram-positive cocci are completely inhibited.

The addition of 100 mg of 4-methylumbelliferyl- β -D-glucuronide to one litre of MacConkey Agar detects the enzyme β -glucuronidase. The cleaved 4-methylumbelliferyl moiety is fluorescent at 366 nm. Thus, colonies of *Escherichia coli* can be detected rapidly in mixed cultures by examining the plate under a UV lamp after overnight incubation at 35°C. However, it should be remembered that other organisms may also be β -glucuronidase positive.

Typical Formula

	grams per litre
Peptone	20.0
Lactose	10.0
Bile salts No. 3	1.5
Sodium chloride	5.0
Neutral red	0.03
Crystal violet	0.001
Agar	15.0

Materials Provided

CM0115B: 500g of MacConkey Agar No.3
 CM0115K: 25kg of MacConkey Agar No.3
 CM0115M: 3 x 3.09kg of MacConkey Agar No.3
 CM0115Q: 5 x 5.15kg of MacConkey Agar No.3
 CM0115R: 2.5kg of MacConkey Agar No.3
 CM0115T: 5kg of MacConkey Agar No.3
 CM0115V: 10 x 2.57kg of MacConkey Agar No.3
 CM0115W: 2 x 5.15kg of MacConkey Agar No.3

500g of dehydrated of MacConkey Agar No.3 yields approximately 9.7L after reconstitution.

Materials Required but Not Supplied

- Inoculating loops, swabs, collection containers
- Incubators
- Quality control organisms
- Petri dish

Storage

- Store product in its original packaging between 10°C and 30°C.
- Keep container tightly closed.
- The product may be used until the expiry date stated on the label.
- Protect from moisture.
- Store away from light.
- Allow reconstituted product to equilibrate to room temperature before use.

Once reconstituted, store media between 2°C and 10°C.

Warnings and Precautions

- Do not inhale. May cause allergy or asthma symptoms or difficulty breathing if inhaled.
- Causes serious eye irritation.
- May cause an allergic skin reaction.
- If on skin wash with plenty of soap and water.
- If in eyes, rinse cautiously with water for several minutes.
- Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists, seek medical advice/attention.
- If inhaled, if breathing is difficult, remove subject to fresh air and keep in a position comfortable for breathing. If experiencing respiratory symptoms, call a POISON CENTER or doctor/physician.
- For in vitro diagnostic use only.
- For professional use only.
- Inspect the product packaging before first use.
- Do not use the product if there is any visible damage to the packaging (pot or cap).
- Do not use the product beyond the stated expiry date.
- Do not use the device if signs of contamination are present.
- It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products.
- Ensure the lid of the container is kept tightly closed after first opening and between use to minimise moisture ingress, which may result in incorrect product performance.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

Specimen should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI) ID 1, ID 16, ID 20, ID 24, B 4, B 11, B 14, B 15, B 17, B 26, B 37 and S 7.

Procedure

Suspend 51.5g in 1 litre of distilled water. Bring to the boil to dissolve completely. Sterilise by autoclaving at 121°C for 15 minutes. Cool to 50°C. Mix well and pour into sterile containers.

Interpretation

Once the medium is reconstituted: *Salmonella virchow*, *Salmonella abony*, *Shigella sonnei* and *Shigella boydii* are indicated by the presence of straw colonies. *Proteus mirabilis* is indicated by the presence of straw colonies with slight or no swarming. *Escherichia coli* is indicated by pink colonies with bile precipitation. Other identification procedures should be performed to confirm findings.

More details at: www.thermoscientific.com/microbiology

Quality Control

It is the responsibility of the user to perform Quality Control testing taking into account the intended use of the medium, and in accordance with any local applicable regulations (frequency, number of strains, incubation temperature etc.).

The performance of this medium can be verified by testing the following reference strains.

Incubation Conditions: 18-24 hours @ 37°C

Positive Controls Inoculum level: 10-100 cfu Colony count is ≥ 70% of the control medium count	
<i>Escherichia coli</i> ATCC® 8739™	0.5 – 6 mm pink colonies bile precipitation
<i>Salmonella Virchow</i> NCTC 5742	1-8 mm straw colonies
<i>Salmonella abony</i> NCTC 6017	1-8 mm straw colonies
<i>Shigella sonnei</i> ATCC® 25931™	1-12 mm straw colonies
<i>Shigella boydii</i> NCTC 11462	1-12 mm straw colonies
<i>Proteus mirabilis</i> ATCC® 29906™	1-8 mm straw colonies, slight/ no swarming
Negative Control Inoculum level: 10 ⁴ -10 ⁶ cfu Incubation Conditions: 18-24 hours @ 30-35°C	
<i>Staphylococcus aureus</i> ATCC® 6538™	No growth

Limitations

Prolonged incubation may lead to confusing results. Do not incubate beyond 48 hours. Test the medium with a laboratory stock strain

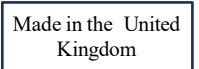
of *Shigella* species which is in the R- phase. R-phase shigellae should grow satisfactorily on MacConkey Agar.

Performance Characteristics

Accuracy has been demonstrated through review of the QC data. Correct detection of microorganisms is confirmed by the inclusion of a well-characterised isolate in the QC processes performed as part of the manufacture of each batch of the devices. The precision of the MacConkey Agar No. 3 (CM0115B/K/M/Q/R/T/V/W) devices was demonstrated by an overall pass rate of 100% obtained for the products over two months of testing (06.05.2022 – 14.07.2022; 10 batches. This shows that the performance is reproducible.

MacConkey Agar No. 3 (CM0115B/K/M/Q/R/T/V/W) device is tested in-house as part of the QC process since the products were launched in 1995. For target organisms, when using 10-100 cfu inoculum of *Escherichia coli*, *Salmonella virchow*, *Salmonella abony*, *shigella sonnei*, *Shigella boydii* and *Proteus mirabilis* and incubating the device at 37°C for 18-24 hours the user can recover organisms with colony size and morphology as listed in this document. When using 10⁴-10⁶ cfu of *Staphylococcus aureus* and incubating the device at 30-35°C for 18-24 hours, the user can expect no growth.

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date
	Keep away from sunlight
	Consult instructions for use or consult electronic instructions for use
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Importer - To indicate the entity importing the medical device into the locale. Applicable to the European Union
	Made in the United Kingdom

ATCC Licensed
Derivative

©2023 Thermo Fisher Scientific Inc. All rights reserved. ATCC and ATCC catalogue marks are a trademark of American Type Culture Collection.

NCTC and NCTC catalogue marks are a trademark of National Collection of Type Cultures.

All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

For technical assistance please contact your local distributor.

Revision information

Version	Date of modifications introduced
3.0	2023-12-18

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Alla övriga varumärken tillhör Thermo Fisher Scientific Inc. och dess dotterbolag.
Oxoid Limited, Wade Road, Basingstoke, RG24 8PW, Storbritannien