www.thermofisher.com

EN

Microbact™ Mineral Oil

REF MB1093A..... 50ml

1. INTENDED USE

Microbact™ Mineral Oil is a reagent for use as an overlay to aid in qualitative procedures for the detection of microorganisms in biochemical tests. The device is used in a diagnostic workflow to aid clinicians in the treatment options for patients suspected of having microbial infections.

The device is for professional use only, it is not automated nor is it a companion diagnostic.

2. PRINCIPLE OF THE TEST

Microbact Mineral Oil is a reagent for use as an overlay in qualitative procedures for the detection of microorganisms in biochemical tests and other methods. When it is used with biochemical tests, the Microbact Mineral Oil is added to certain wells of the identification panels in order to create a micro-anaerobic environment that supports the biochemical reactions involved. Result interpretation is based on published reference methodology.

In other qualitative tests, the Microbact Mineral Oil may also be used as a sealant to create an anaerobic environment for biochemical tests such as decarboxylase and oxidation and fermentation reactions or to avoid any evaporation that may occur during preparation of cultures for further analysis.

3. KIT CONTENTS, PREPARATION FOR USE AND STORAGE

Microbact Mineral Oil..... 50ml
One dropper bottle containing paraffin oil.

The expiration of the product is stated on the package label.

This product is ready for use and no further preparation is necessary. Store product in its original container at 10-30°C until expiry date.

4. PRECAUTIONS

This product is for *in vitro* diagnostic use and should be used by properly trained individuals. Precautions should be taken against the dangers of microbiological hazards by properly sterilizing specimens, containers, and media after use. Directions should be read and followed carefully.

Avoid contact with skin, eyes or clothing. Wear personal protective equipment/face protection. Ensure adequate ventilation. Avoid ingestion or inhalation.

Discard used material into a suitable waste container or disinfectant.

Do not use the product beyond its stated expiry date.

Do not use if there is evidence of contamination, or other signs of deterioration.

Do not use if packaging is damaged.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

In the event of malfunction do not use device.

5. SPECIMEN COLLECTION, STORAGE, AND TRANSPORT

Specimens should be collected and handled following recommended guidelines.

6. PROCEDURE

Refer to procedure notes with associated biochemical tests for details on use.

7. SYMBOL LEGEND

REF	Catalogue Number
IVD	In Vitro Diagnostic Medical Device
	Consult instructions for use or consult electronic instructions for use
	Temperature Limitations (Storage temp.)
	Do not use if package is damaged
	Not for near patient testing
LOT	Batch Code (Lot Number)
	Use By (Expiration Date)
	To indicate the entity importing the medical device into the locale. Applicable to the European Union
EC REP	Authorized representative in the European Community/European Union
UK CA	UK Conformity Assessed
CE	European Conformity Assessment
Made in Australia	Made in Australia
	Manufacturer

For technical information contact your local distributor.

Oxoid Ltd Wade Road Basingstoke
Hampshire, RG24 8PW UK

Version	Date of modifications introduced
1.0	December 2023 Initial document

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