www.thermofisher.com

Neomycin Selective Supplement

[REF] SR0163H

This instructions for use (IFU) document is intended to be read in conjunction with the IFU for Wilkins-Chalgren Anaerobe Agar Base, product code: CM0619B.

Intended Use

Neomycin Selective Supplement (SR0163H) is used in the preparation of Anaerobe basal medium for the selective isolation of *Clostridium* species from clinical samples. It can also be used in the preparation of Wilkins-Chalgren medium for the isolation of *Bacteroides* and *Clostridium* species from clinical samples.

The device is for professional use only, is not automated and nor is it a companion diagnostic.

Summary and Explanation

Anaerobic bacteria are important pathogens that can cause a variety of infections in humans. The site of anaerobic infection is commonly the site of normal colonization. The spectrum of infections ranges from superficial abscesses to life-threatening infections. Anaerobic microorganisms, including pathogens such as *Clostridium perfringens*, *Bacteroides fragilis* and *Fusobacterium nucleatum*, can be highly infectious and can cause life-threatening diseases. *Fusobacterium necrophorum* has been linked to infections such as necrobacillosis and Lemierre's disease.¹ Anaerobes are endogenous in nature, as they are a primary component of the flora of mucus membranes and gastrointestinal tract. However, they are often difficult to isolate and recognise due to their fastidious nature and overgrowth of facultative anaerobes such as coliforms. This can lead to delays in diagnosis, and therefore delays in determining appropriate therapy which may lead to clinical failures.²

Principle of Method

Neomycin is an aminoglycoside antibiotic which has bactericidal activity against many Gram-negative facultative anaerobes and some staphylococci. This is added to the medium to inhibit accompanying flora.

Typical Formula

Neomycin sulphate 150.0 mg

Materials Provided

SR0163H: 10x freeze dried vials, each for supplementing 2L of medium.

Materials Required but Not Supplied

- Wilkins-Chalgren Anaerobe agar CM0619B/CM0619T/CM0619W
- Blood Agar Base No.2 CM0271B
- Inoculating loops, swabs, collection containers.
- Petri dishes.
- Sterile distilled water
- Pipettes

Storage

- Store product in its original packaging between 2°C and 8°C.
- Keep container tightly closed.
- The product may be used until the expiry date stated on the label.
- Protect from moisture.
- Store away from light.
- Allow vials to equilibrate to room temperature before use.

Warnings and Precautions

- May cause an allergic skin reaction.
- May cause allergy or asthma symptoms or breathing difficulties if inhaled.
- Suspected damage of unborn child.
- Wear protective gloves/protective clothing/eye protection/face protection.
- IF ON SKIN: Wash with plenty of soap and water while removing all contaminated clothes and shoes. Get medical attention.
- If skin irritation or rash occurs: Get medical advice/attention.
- IF IN EYES: Rinse thoroughly with plenty of water, also under the eyelids. Get medical attention immediately if symptoms occur.
- In case of inadequate ventilation wear respiratory protection.
- IF INHALED: Remove to fresh air. Get medical attention. Oxygen or artificial respiration if needed. Consult a physician if necessary.
- If experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician
- Avoid release to the environment.
- IF INGESTED: Do NOT induce vomiting. Get medical attention.
- DO NOT USE SUPPLEMENTS IF YOU ARE PREGNANT OR CONTEMPLATING PREGNANCY.
- Each vial is for single use. Do not re-use.
- For in vitro diagnostic use only.
- For professional use only.
- Inspect the product packaging before first use.
- Do not use the product if there is any visible damage to the packaging or vials.
- Do not use the product beyond the stated expiry date.
- Do not use the device if signs of contamination are present.
- Do not use the device if the colour has changed or there are other signs of deterioration.

It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

There are no specimen collection and/or transport materials provided with this device. Specimens should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI) B 11, B 17, ID 14, ID 25 and Q 5.

Procedure

- To one vial aseptically add 2 ml of sterile distilled water and mix gently to dissolve the contents completely.
- Avoid frothing.
- Add the contents to 2.0 litres of sterile Wilkins-Chalgren Anaerobe Agar (CM0619B/T/W) cooled to 50°C containing 5% sterile defibrinated horse blood (SR0050C) **OR** to Blood Agar Base No.2 (CM0271B) cooled to 50°C, containing 5% sterile defibrinated horse blood (SR0050C) **OR** to Anaerobe Basal Medium (CM0972B).

Bibliography

- Public Health England. 2015b. 'Identification of Anaerobic Gram-Negative Rods'. UK Standards for Microbiology Investigations ID 25 (2). <https://www.gov.uk/government/publications/smi-id-25-identification-of-anaerobic-gram-negative-rods>.
- Brook, I. and Frazier, E. H., 1998. 'Aerobic and anaerobic microbiology of infection after trauma'. American Journal of Emergency Medicine 16(6):585-591. [https://doi.org/10.1016/S0735-6757\(98\)90225-X](https://doi.org/10.1016/S0735-6757(98)90225-X)

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date
	Keep away from sunlight
	Do not re-use
	Consult instructions for use or consult electronic instructions for use
	Contains sufficient for <n> tests
	Do not use if packaging damaged and consult instructions for use

	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Made in the United Kingdom

©2022 Thermo Fisher Scientific Inc. All rights reserved. All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Oxoid Limited, Wade Road, Basingstoke, RG24 8PW, UK

For technical assistance please contact your local distributor.

Revision information

Version	Modifications introduced
1.0	2022-10-27 New document (LIVE)

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Oxoid Limited, Wade Road, Basingstoke, RG24 8PW, Reino Unido