



AmnioTest™ offers the most efficient, reliable, and simple to use point-of-care screen test for the detection of amniotic fluid in near-term patients. Along with a simple Q.C. regimen, AmnioTest™ is an invaluable tool for OB-GYN and L+D clinics.

AmnioTest™ Product Availability

CODE	PRODUCT	SIZE
PL.901	AmnioTest™	100 Swabs
PL.902	Colour Interpretation Cards	50 Cards
PL.910	Buffer Solution pH 5.0	2.0 mL
PL.911	Buffer Solution pH 5.5	2.0 mL
PL.912	Buffer Solution pH 6.0	2.0 mL
PL.913	Buffer Solution pH 6.5	2.0 mL
PL.914	Buffer Solution pH 7.0	2.0 mL
PL.915	Buffer Solution pH 7.5	2.0 mL

Pro-Lab Diagnostics U.K.
 7 Westwood Court
 Neston, South Wirral, Cheshire
 CH64 3UJ
 United Kingdom
 Tel: 0151 353 1613
 Fax: 0151 353 1614
 E-Mail: uksupport@pro-lab.com

Pro-Lab Diagnostics Canada
 20 Mural Street, Unit #4
 Richmond Hill, ON, L4B 1K3
 Canada
 Toll Free: 1-800-268-2341
 Tel: (905) 731-0300
 Fax: (905) 731-0206
 E-Mail: support@pro-lab.com

Pro-Lab Diagnostics U.S.A.
 9701 Dessau Road, Suite #802
 Austin, TX, 78754-3941
 U.S.A.
 Toll Free: 1-800-522-7740
 Tel: (512) 832-9145
 Fax: 1-800-332-0450
 E-Mail: ussupport@pro-lab.com



www.pro-lab.com

AmnioTest™ Rapid Amniotic Fluid Swab Test

AmnioTest™ is designed to assist in the detection of rupture of the fetal membrane. It offers the most efficient, cost effective, reliable, and simple to use point-of-care screen test to aid in the detection of amniotic fluid in near-term patients. Along with a simple quality control regimen, AmnioTest™ is an invaluable tool for OB-GYN and L+D clinics.

Simple Test Procedure


1. "Peel"

Remove sterile swab from individual wrapper by peeling open the AmnioTest™ sleeve. Note: Do not allow swab tip to come in contact with anything before test is performed.



2. "Swab"

Insert swab so that nitrazine tip gently comes in contact with the upper cervical area for 10-15 seconds. A speculum may be used during this phase of the procedure to limit swab exposure to anything that may affect pH (i.e. urine, bacteria).

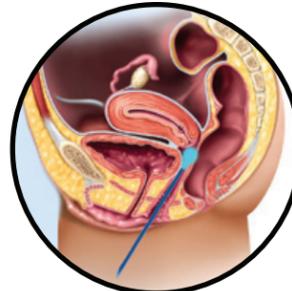

3. "Read"

After carefully removing swab, simply compare the swab tip colour to the interpretation card supplied by Pro-Lab and record the corresponding pH level. Discard swab and return box to storage area being sure to close lid properly. Exposure to light or humidity may affect performance of the product.



Be sure to review the technical insert provided by Pro-Lab if you are unsure about any step in the procedure, including interpretation of results.

PRODUCT FEATURES

- Rapid, safe, easy to use
- Test results in 15 seconds
- New and improved packaging, featuring "Easy Peel" opening
- 100 individually wrapped swabs per box
- Sterile for patient safety
- A speculum is not required
- Easy to QC: Once per lot number
- QC Buffer solutions available
- Clinical education tutorials available by request

Result Interpretation

AmnioTest™		Colour Card
	pH 5.0	 A
	pH 5.5	
	pH 6.0	
	pH 6.5	 C
	pH 7.0	
	pH 7.5	

Canada: 800 268 2341 Fax: 905 731 0206
 U.S.A.: 800 522 7740 Fax: 800 332 0450
 U.K.: 0151 353 1613 Fax: 0151 353 1614

20 Maral Street, Unit 4
 Richmond Hill, ON L4B 1K3
 Tel: (905) 731-2300



For illustrative purposes only

Visit the official AmnioTest™ website at:

www.amniotest.com

