remelTM

EN

RapID Nitrate A Reagent & RapID Nitrate B Reagent

REF R8309003..... 15 ml/Bottle

REF R8309004..... 15 ml/Bottle

1. INTENDED USE

RapIDTM Nitrate A and RapID Nitrate B are reagents to be used with the RapID manual identification systems.

The devices are for professional use only, is not automated nor is it a companion diagnostic.

2. SUMMARY AND EXPLANATION

Nitrate is a source of nitrogen for many bacteria. The ability to reduce nitrate to nitrite is characteristic of the family Enterobacteriaceae.¹ Non-fermentative Gram-negative bacilli and other miscellaneous Gram-negative and Gram-positive bacteria vary in their ability to reduce nitrate. Nitrate reduction results in several end-products depending on the bacterial species being tested.²

3. PRINCIPLE

The nitrate reduction test is a two-step process. RapID Nitrate A Reagent is added to acidify the medium followed by RapID Nitrate B Reagent which reacts with nitrite to produce a red colour. Some organisms further reduce nitrites to form nitrogen gas or ammonia. If no colour develops following the addition of both reagents, the organism is either not capable of reducing nitrate or it has further reduced nitrite to free nitrogen gas.³

4. REAGENTS

RapID Nitrate A Reagent..... 15ml/Bottle

Sulfanilic Acid (CAS 121-57-3)8.0 g
Glacial Acetic Acid (CAS 64-19-7) 280.0 ml
Demineralized Water (CAS 7732-18-5)..... 720.0 ml

RapID Nitrate B Reagent 15ml/Bottle

N,N-dimethyl-1-naphthylamine (CAS 86-56-6).....6.0 g
Glacial Acetic Acid (CAS 64-19-7) 280.0 ml
Demineralized Water (CAS 7732-18-5)..... 720.0 ml

5. PRECAUTIONS

DANGER! Causes severe digestive and respiratory tract burns. Causes severe eye and skin burns.

This product is for *in vitro* diagnostic use and should be used by properly trained individuals. Precautions should be taken against the dangers of microbiological hazards by properly sterilizing specimens, containers and media after use. Directions should be read and followed carefully. Refer to Material Safety Data Sheet (SDS) on company website for additional information.

Any serious incident that has occurred in relation to the

device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

In the event of malfunction do not use device.

Nitrate A
Composition / information on ingredients
Acetic acid 64-19-7
Sulfanilic acid 121-57-3

Nitrate B
Composition / information on ingredients
Acetic acid 64-19-7
N,N-dimethyl-1-naphthylamine - 86-56-6

DANGER

US ONLY

US ONLY

US & EU

H226	Flammable liquid and vapor
H314	Causes severe skin burns and eye damage
H335	May cause respiratory irritation
P260	Do not breathe dust/fume/gas/mist/vapors/spray
P264	Wash face, hands and any exposed skin thoroughly after handling
P280	Wear protective gloves/protective clothing/eye protection/face protection
P271	Use only outdoors or in a well-ventilated area
P210	Keep away from heat/sparks/open flames/hot surfaces. - No smoking
P233	Keep container tightly closed
P240	Ground/bond container and receiving equipment
P241	Use explosion-proof electrical/ventilating/lighting/equipment
P242	Use only non-sparking tools
P243	Take precautionary measures against static discharge
P235	Keep cool
P310	Immediately call a POISON CENTER or doctor/physician
P304+P340	IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing
P303+P361 +P353	IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower
P363	Wash contaminated clothing before reuse
P305+P351 +P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing
P301+P330 +P331	IF SWALLOWED: Rinse mouth. DO NOT induce vomiting
P370+P378	In case of fire: Use CO2, dry chemical, or foam for extinction
P405	Store locked up
P403+P233	Store in a well-ventilated place. Keep container tightly closed
P501	Dispose of contents/container to an approved waste disposal plant

Emergency Telephone Number

INFOTRAC - 24 Hour Number: 1-800-535-5053

Outside of the United States, call 24 Hour Number: 001-352-323-3500 (Call Collect)

6. STORAGE

This product is ready for use and no further preparation is necessary. Store product in its original container at 2-8°C until used.

7. PRODUCT DETERIORATION

This product should not be used if (1) the colour has changed, (2) the expiration date has passed, or (3) there are other signs of deterioration.

8. SPECIMEN COLLECTION, STORAGE, AND TRANSPORT

Specimens should be collected and handled following recommended guidelines.³

9. MATERIALS REQUIRED BUT NOT SUPPLIED

(1) Loop sterilization device, (2) Inoculating loop, swabs, collection containers, (3) Incubators, alternative environmental systems, (4) Supplemental media, (5) Quality control organisms.

10. QUALITY CONTROL

All lot numbers of RapID Nitrate A Reagent and RapID Nitrate B Reagent have been tested and found to be acceptable. Quality control organisms should be tested according to the guidelines in the RapID Identification System in use or according to established laboratory quality control procedures. If aberrant quality control results are noted, patient results should not be reported.

11. BIBLIOGRAPHY

1. Ewing, W.H. 1986. Edwards & Ewing's Identification of Enterobacteriaceae. 4th ed. Elsevier, New York, NY.

2. MacFaddin, J.F. 2000. Biochemical Tests for the Identification of Medical Bacteria. 3rd ed. Lippincott Williams & Wilkins, Philadelphia, PA.

3. Carroll, K C, M A Pfaller et al., 2019. Manual of Clinical Microbiology: Twelfth Edition (ASM Books)

12. PACKAGING

REF R8309003, RapID Nitrate A Reagent 15 ml/Bottle

REF R8309004, RapID Nitrate B Reagent 15 ml/Bottle

13. SYMBOL LEGEND

	Catalogue Number
	In Vitro Diagnostic Medical Device
	Consult Instructions for Use (IFU)
	Temperature Limitations (Storage temp.)
	Do not use if package is damaged
	Not for near patient testing
	Batch Code (Lot Number)
	Use By (Expiration Date)
	Importer
	Authorized representative in the European Community
	UK Conformity Assessed
	European Conformity Assessment
	Manufacturer

For technical information contact your local distributor.

©2022 Thermo Fisher Scientific Inc. All rights reserved.

All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

CAS (Chemical Abstracts Service Registry No.)

 12076 Santa Fe Trail Drive, Lenexa, KS 66215, USA

www.thermofisher.com/microbiology
Tel: (800) 255-6730 • International: (913) 888-0939

www.oxid.com/IFU
Europe +800 135 79 135 • US 1 855 2360 190
CA 1 855 805 8539 • ROW +31 20 794 7071

Version	Date of modifications introduced
IFU8309003	March 2023 Addition of Importer symbol to Section 13

Printed in the UK

RapID™ Nitrate A Reagent and RapID Nitrate B Reagent

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11. LITERATŪRA

- 1. Ewing, W.H. 1986. Edwards & Ewing’s Identification of Enterobacteriaceae. 4th ed. Elsevier, New York, NY.
- 2. MacFaddin, J.F. 2000. Biochemical Tests for the Identification of Medical Bacteria. 3rd ed. Lippincott Williams & Wilkins, Philadelphia, PA.
- 3. Carroll, K C, M A Pfaller et al., 2019. Manual of Clinical Microbiology: Twelfth Edition (ASM Books)

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remelTM PL RapIDTM Nitrate A Reagent and RapID Nitrate B Reagent

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11. BIBLIOGRAFIE

1. Ewing, W.H. 1986. Edwards & Ewing's Identification of Enterobacteriaceae. 4th ed. Elsevier, New York, NY.
2. MacFaddin, J.F. 2000. Biochemical Tests for the Identification of Medical Bacteria. 3rd ed. Lippincott Williams & Wilkins, Philadelphia, PA.
3. Carroll, K C, M A Pfaller et al., 2019. Manual of Clinical Microbiology: Twelfth Edition (ASM Books)

12. AMBALARE

REF R8309003, RapID Nitrate A Reagent 15 ml/flacon

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12076 Santa Fe Trail Drive, Lenexa, KS 66215, SUA

www.thermofisher.com/microbiology

Tel.: (800) 255-6730 • Internațional: (913) 888-0939

www.oxoid.com/IFU

Europa +800 135 79 135

CA 1 855 805 8539

• SUA 1 855 2360 190

• Alte țări +31 20 794 7071

remelTM SK RapIDTM Nitrate A Reagent and RapID Nitrate B Reagent

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©2022 Thermo Fisher Scientific Inc. Všetky práva vyhradené.